THUASNE

Ligaflex® Rhizo

fr	Attelle de stabilisation du pouce	4
en	Stabilising thumb brace	5
de	Orthese zur Stabilisierung des Daumens	6
nl	Stabilisatiebrace voor de duim	7
it	Tutore stabilizzatore per pollice	8
es	Férula estabilizadora del pulgar	9
pt	Tala de estabilização do polegar	10
da	Stabiliserende skinne til tommelfingeren	11
fi	Stabiloiva peukalotuki	12
sv	Stabiliseringsskena för tummen	14
el	Νάρθηκας ακινητοποίησης του αντίχειρα	15
cs	Ortēza pro stabilizaci palce	16
pl	Orteza stabilizująca kciuka	17
lv	Īkšķa stabilizācijas saite	18
lt	Nykščio stabilizavimo įtvaras	19
et	Pöidla stabiliseerimislahas	20
sl	Opornica za stabilizacijo palca	21
sk	Ortēza na znehybnenie palca	22
hu	Hüvelykujj-stabilizáló sín	23
bg	Шина за стабилизация на палеца	25
ro	Atelă stabilizatoare pentru degetul mare	26
ru	Стабилизирующий ортез для большого пальца	27
hr	Stabilizacijska ortoza za palac	28
zh	拇指稳定夹板	29
ar	جبيرة لتثبيت الإبهام	31

	1	13 - 17 cm
	2	17 - 23 cm

fr	Tour de poignet
en	Wrist circumference
de	Handgelenkumfang
nl	Polsomtrek
it	Circonferenza del polso
es	Contorno de la muñeca
pt	Perímetro do punho
da	Håndleddets omkreds
fi	Ranteen ympärymitta
sv	Handledens omkrets
el	Περίμετρος καρπού
cs	Obvod zápěstí
pl	Obwód nadgarstka
lv	Plaukstas locītavas apkārtmērs
lt	Riešo apimtis
et	Randme ümbermõõt
sl	Obseg zapestja
sk	Obvod zápästia
hu	Csukló kerülete
bg	Обиколка на китката
ro	Circumferința încheieturii
ru	Обхват запястья
hr	Obujam ručnog zgloba
zh	腕围
ar	محيط الرسغ

fr	Stabilisation
en	Stabilisation
de	Stabilisierung
nl	Stabilisatie
it	Stabilizzazione
es	Estabilización
pt	Estabilização
da	Stabilisering
fi	Stabiloi
sv	Stabilisering
el	Σταθεροποίηση
cs	Stabilizace
pl	Stabilizacja
lv	Stabilizēšana
lt	Stabilizavimas
et	Stabiliseerimine
sl	Stabiliziranje
sk	Stabilizácia
hu	Stabilizálás
bg	Стабилизация
ro	Stabilizare
ru	Стабилизация
hr	Stabilizacija
zh	穩定
ar	الاستقرار

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STABILISING THUMB BRACE

Description/Destination

This device is a thumb brace providing joint stabilisation.

The device is intended only for the treatment of the indications listed and for patients whose measurements correspond to the sizing table.

Composition

Textile components: polyamide - polyester - elastane - polyurethane.

Rigid components: stainless steel.

Properties/Mode of action

Thumb stabilisation via a metallic wire stay.

Aerated, foam lined fabric.

Two self-fastening straps for easy fitting of the product.

Indications

Thumb osteoarthritis (including rhizarthrosis).

Immobilisation in case of pain or trauma of the thumb.

Contraindications

Do not use the product if the diagnosis has not been confirmed.

Do not use in the event of known allergy to any of the components.

Do not apply the product in direct contact with broken skin.

Precautions

Verify the product's integrity before every use.

Do not use the device if it is damaged.

Choose the appropriate size to fit the patient, referring to the size chart.

It is recommended that a healthcare professional supervises the first application.

Strictly comply with your healthcare professional's prescription and recommendations for use.

For hygiene and performance reasons, do not re-use the product for another patient.

In the event of discomfort, significant hindrance, pain, variation in limb volume, abnormal sensations or change in colour of the extremities, remove the device and consult a healthcare professional.

Do not use the device in case of application of certain products on the skin (creams, ointments, oils, gels, patches...).

It is recommended to adequately tighten the device to achieve support/immobilisation without restricting blood circulation.

Undesirable side-effect

This device can cause skin reactions (redness, itching, burns, blisters...) or wounds of various degrees of severity.

Any serious incidents occurring related to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

Instructions for use/Application

- 1 - Open the wrist straps.
- 2 - Slip the thumb into the splint.
- 3 - Adjust the wrist strap, pulling it through the tightening loop.
- 4 - Adjust the thumb strap.
- 5 - Place the black seam in the middle of the palm of the hand.

Care/Maintenance

Product can be washed in accordance with the conditions shown on this notice and on the label. Machine washable at 30 °C (delicate programme). Close the self-fastening tabs before washing. Do not use detergents, fabric softeners or aggressive products (products containing chlorine, etc.). Do not tumble-dry. Do not iron. Do not dry clean. Squeeze out excess water. Dry away from any direct heat source (radiator, sun, etc.).

Storage

Store at room temperature, preferably in the original packaging.

Disposal

Dispose of in accordance with local regulations.

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©Thuasne - 2020101 (2021-02)